

# QUICK START GUIDE

Let's get started!

Don't miss important updates! Register your IQbuds™ through the app.



**IQbuds**

**IQbuds**


BOOST

## IN THE BOX:


IQbuds™ - Left and Right

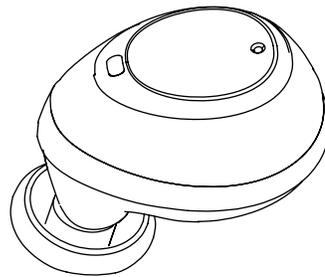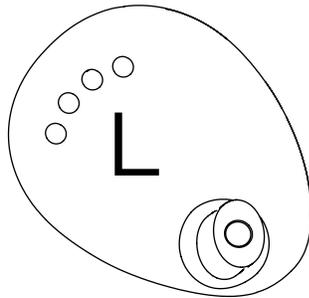


Charging/carrying case

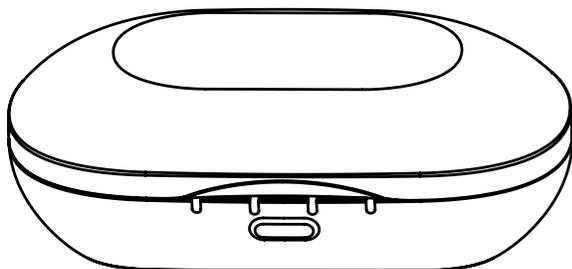

Ear tips

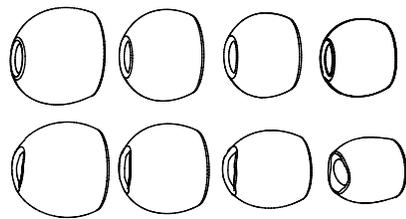


USB cord

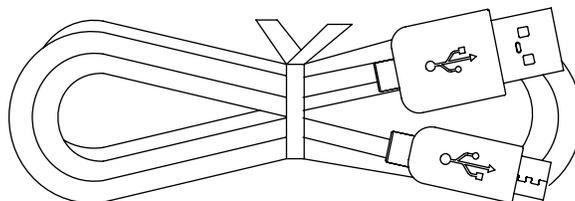

# CHARGING

## STEP 1


There are two devices to be charged - the charging case and the earbuds.

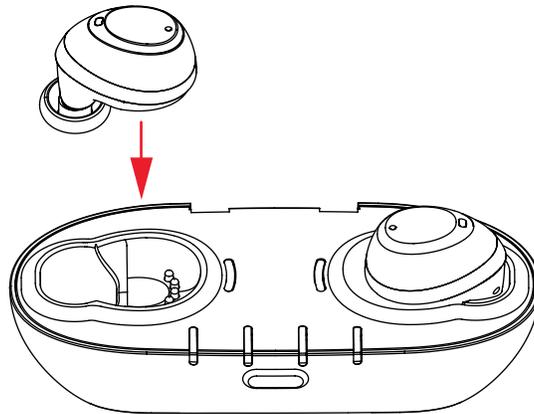

Place your earbuds in the case, ensuring the gold contacts are touching the pins, and close the case to begin charging.

Two red lights on the case will illuminate for a few seconds to confirm the earbuds are charging.

## STEP 2

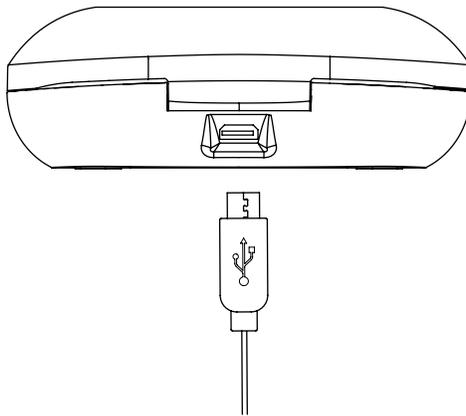


Connect the case to a USB power source (e.g. to a computer or a charging plug) to charge both the case and the earbuds.

To fully charge case and earbuds for first use	4 hours
To fully charge case	2.5 hours
To fully charge earbuds from case without USB power source	1.5 hours

## CHARGING INDICATORS


There are two devices to be charged - the charging case and the earbuds.

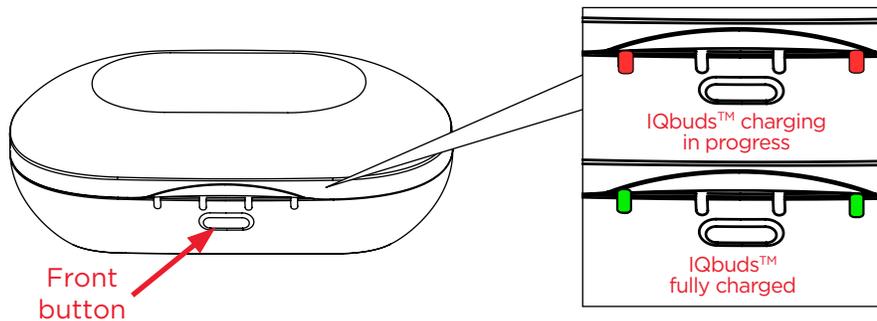

### Earbuds

When your earbuds are in the case and you close the lid, the indicator lights will display the charging status of the earbuds, as shown above.

If you missed it, press the **front button** for less than 2 seconds ('short press') to display the earbud charging status again.

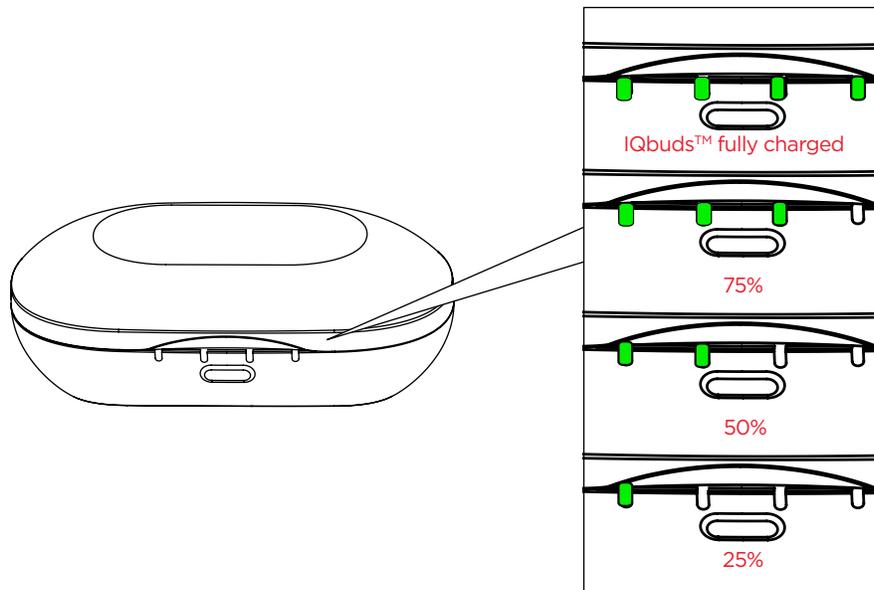


### Charging case

When the case is empty and you open it, the indicator lights will show the charging status of the case, as shown above.

When your earbuds are in the case with the lid closed, you can see the charging status of the case by pressing the button for more than 2 seconds ('long press').


## FIT

### STEP 1

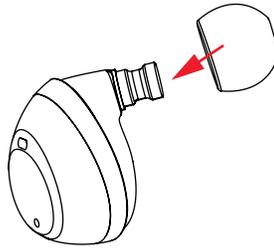


Gently push the rubber ear tips onto the earbuds as shown. Try the different sizes and shapes of ear tips to get the best fit.

### STEP 2

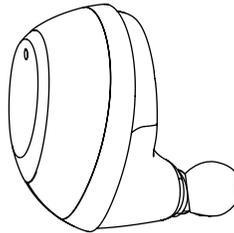

Insert the earbud into your ear with the tip pointing down.

### STEP 3

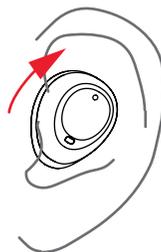

Rotate the earbud toward the back of your head. It should be seated snugly within the ear at a slight backwards angle.

▶ Ear tips should provide a sound-proof seal in the ear without being uncomfortably tight.

## GET THE APP



## PAIR WITH YOUR DEVICE

### STEP 1

Remove your IQbuds from the charging case.

### STEP 2

Insert your IQbuds in your ears.

### STEP 3

Press and hold the tap-touch sensor (either L or R) for 5 seconds ('very long tap'). The announcer will say 'Pairing'. When they have paired with your mobile device, the announcer will say 'Connected'.



You can check if your earbuds are connected by viewing the Bluetooth settings on your mobile device.


### STEP 4

Open the app and follow the tutorials to adjust your earbuds to suit your hearing levels and your environment.

# CONTROLS

## LEFT


Left tap-touch sensor

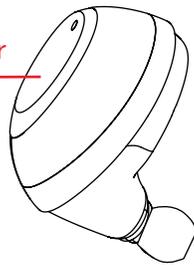

	Music	Phone call
Tap	Play/pause	Answer call / Mute call
Double-tap	Activate Siri/ Google Now	
Long tap		Reject/ hang up
5s Very long tap	Activate Pairing mode	

## RIGHT

Right tap-touch sensor

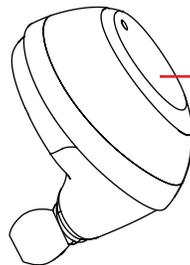

	Environment
Tap	Toggle SINC on/off
Double- tap	Activate Siri/Google Now
Long tap	Switch between favourite locations
5s Very long tap	Activate Pairing mode

TECHNICAL SUPPORT  
Register your IQbuds™ and use the app to contact technical support.



FULL MANUAL & ONLINE SUPPORT  
visit [www.nuheara.com/support](http://www.nuheara.com/support) to view help documents, explainer videos and FAQs.